

UNICSKIN

YOUR EFFECTIVE SOLUTION

UNICWHITE SMILE GEL

HOME TEETH WHITENING



**PROFESSIONAL TEETH WHITENING AT HOME.
NO PEROXIDES. NO SENSITIVITY.**



SENSITIVITY-FREE
FORMULA



FAST,
VISIBLE RESULTS



PROTECTS TOOTH
ENAMEL



GENTLE ON
GUMS

USER GUIDE



UNICWHITE SMILE GEL

ENAMEL-FRIENDLY WHITENING GEL

Dental Whitening Gel formulated with 18% PAP, a next-generation active ingredient that **removes stains without releasing free radicals or damaging the enamel**. Its **peroxide-free formula** helps achieve a visibly whiter smile **without sensitivity**.

PAP (Phthalimidoperoxycaproic Acid) represents a new era in whitening technology. Unlike peroxide-based gels—common in many cosmetic products but restricted to 0.1% by regulation—PAP allows for a more effective and safer action at higher concentrations (up to 20%), delivering visible results in less time and without compromising enamel.

Activated by the blue light of the UNICLED WHITE SMILE 5.0 device, the gel **enhances its performance on stains caused by coffee, wine, or tobacco**.

It can be used as a standalone treatment or as a complement to extend the effects of professional whitening, thanks to an advanced formula that blends technology, efficacy, and care.

Enriched with **peppermint essential oil**, it offers a mild minty flavor and a refreshing sensation during and after each application. The treatment includes **16 single-dose ampoules for 1 month of use, with sessions lasting only 10 minutes**.

♥ BENEFITS

- **Effective Whitening without Peroxide or Sensitivity.**
- **Compatible** with **Sensitive Teeth and Gums.**
- **Vegan & Biodegradable** Formula.
- Single-Dose, **Travel-Friendly Format.**
- **Visible Results from the Second Week.**

🧪 PROVEN RESULTS

UP TO
8 SHADES WHITER
IN JUST 4 WEEKS!

*Results proven in independent clinical studies. Whitening intensity may vary depending on the natural shade of the teeth.


HOW TO USE

For optimal results, follow these 4 simple steps:



STEP 1

With clean teeth, apply the contents of one UNICWHITE SMILE GEL ampoule evenly onto the mouthpiece of the device. Make sure to cover the entire surface that will come into contact with your teeth.

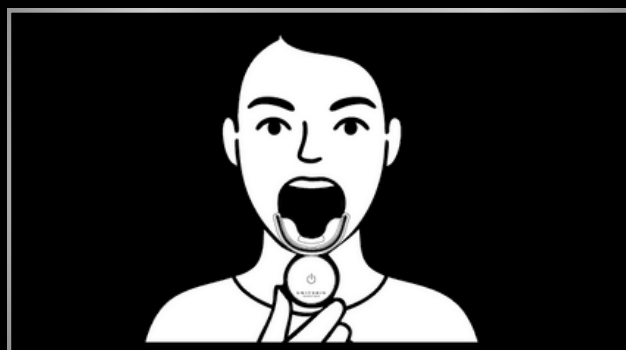

STEP 2

Insert the device into your mouth, ensuring it is properly positioned, and press the power button to start the treatment.


UNICSKIN

YOUR EFFECTIVE SOLUTION



STEP 3

Let it work for 10 minutes. The device is wireless, so you can go about other activities while it operates. It will turn off automatically once the session is complete.

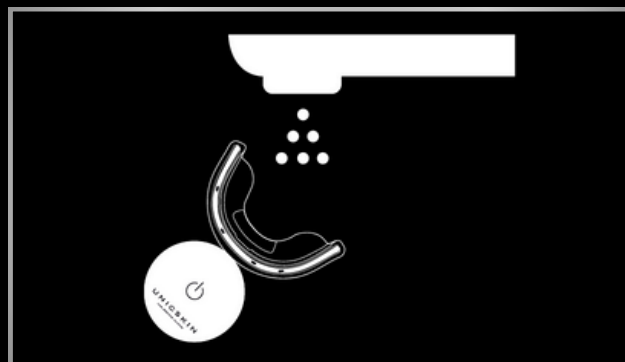

STEP 4

Remove the device, rinse your mouth, and carefully clean the mouthpiece after each use. You can rinse it with water and dry it with a soft cloth. Avoid getting the body of the device wet to prevent damage.

Once the device is clean, place it back into its case for charging. The UV-C sterilization system will automatically activate 5 seconds after the lid is closed; a light will turn on to indicate the process has started.

For optimal results, use 4 times per week for 4 weeks. To maintain your results, it is recommended to repeat the full treatment every 2-3 months.

FREQUENT ASKED QUESTIONS

What active ingredients does the gel contain?

The gel contains 18% PAP (Phthalimidoperoxycaproic Acid), a next-generation whitening agent that removes stains without damaging enamel. It is peroxide-free and enriched with mint extract for a fresh sensation.

Is it safe for sensitive teeth?

Yes. The formula is specifically designed to prevent sensitivity, as it contains no peroxides or abrasive ingredients. It is suitable for people with weak enamel or sensitive gums.

Can it be used without the LED device?

It is not recommended. The gel is formulated to be activated by the blue light of the UNICLED WHITE SMILE 5.0 device, which boosts the effectiveness of PAP. Without the light, whitening results may be significantly reduced.

How many times a week should I use it?

For best results, it is recommended to use it 4 times per week for 4 consecutive weeks. After that, the full treatment can be repeated every 2-3 months for maintenance.

Does the gel affect veneers, crowns, or implants?

No. The gel only works on natural tooth enamel. It does not alter the color of restorative materials such as porcelain, composite, or zirconia.

Can I store the ampoules after opening them?

No. Each ampoule is single-use. It should be discarded immediately after application to ensure hygiene and product effectiveness.

Does it have a flavor?

Yes, the gel is mint-flavored, providing a refreshing sensation in the mouth during and after the treatment.

PRECAUTIONS FOR USE

- Do not apply the gel to the gums or swallow the product.
- Do not wet the body of the device. Only the mouthpiece may be cleaned with water and dried with a soft cloth.
- Do not use if you wear fixed braces (metal brackets), retainers, or removable dental appliances during the session.
- Consult your dentist before use if you have implants, veneers, crowns, or severe dental sensitivity.
- Do not use if you have ulcers, wounds, open sores, active infections in the mouth, or have recently undergone dental procedures.
- Do not use if you are pregnant, planning to become pregnant, or breastfeeding. The device has not been tested under these conditions, so the risk to the mother or baby is unknown.
- Do not use if you suffer from any photosensitive condition (natural or drug-induced), including lupus erythematosus, photosensitive eczema, or albinism.
- Do not use if you are taking medications that may cause photosensitivity (such as certain antibiotics, chemotherapy drugs, or diuretics).
- Do not use if you suffer from migraines or light-induced headaches.
- Do not share the device with others for hygiene reasons.
- Do not use on individuals under 16 years of age.
- Keep out of reach of children and store in a dry place, away from heat sources or direct sunlight.

If you have any concerns about your health or medications you are taking, consult your dentist or physician before starting treatment.

CUSTOMER SERVICE

If you have any questions or comments you'd like to share with us, please write to us at the following email address: customerservices@unicskin.com.

UNICSKIN

YOUR EFFECTIVE SOLUTION

GLOBAL EXPERTS IN TECH-BEAUTY & AT-HOME CARE

SCIENCE-PROVEN, EFFECTIVE RESULTS

SUSTAINABILITY & 92%+ NATURAL ORIGIN FORMULAS

INNOVATIVE LUXURY WITHIN YOUR REACH
